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GB USE WARNINGS

- The VACUMED® Urine tube is for the collection, transport and storage of urine samples for urine culture and urinalysis in the laboratory.
- VACUMED® Urine tubes are made of PET with a pre-defined vacuum for exact draw volumes. The tube interior is sterile. The evacuated tubes can contain a stabilizer to preserve the urine sample at 20 -25°C for up to 36 hours.
- Store tubes at 4 - 25° C (40 - 77° F)
- Avoid exposure to direct sunlight. Exceeding the maximum or undershooting the minimum recommended storage temperature may lead to impairment of the tube quality (e.g. accelerated vacuum loss).
- Allow the tubes to fill until the vacuum has been used up and filling stops automatically. This guarantees the correct (pre-defined) urine to additive ratio. Significant under-filling of the tubes can, depending on the analysis, lead to deterioration of test results. Furthermore urine culture results might be influenced, leading to erroneous test results.
- For evacuated tubes with stabilizer gently invert the tubes 8-10 times, wait for 2-5 minutes, and invert 5-10 times again
- Urine tubes are recommended to be centrifuged at 400g for a period of 5 minutes at 20-25 °
- For tubes with stabilizer the system does not permit determination of the pH value.
- Do not use the product if it is expired or damaged. Do not use the product for any other use which would be considered improper. Store in a dry place, away from sources of heat and direct sunlight. The device is disposable, the reuse of the device will lose all performances and features. Dispose the product in accordance with current legislation in force.

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