

Instruction for Use Pediatric urine bag			Prepared by	Li Xiuren
			Checked by	Zhou Ting
Doc. No.	KB/CE01-08		Approved by	Zhou Hongbao
Effective date	2022.03.01	Ver. B/0	Page No.	Page 1 of 2

Instruction for Use

[Product name]: Pediatric urine bag

[Specification]: 100ml, 200ml

[Materials]: Body bag (PE film), double layer adhesive (water-soluble hot melt adhesive), protecting paper (glossy paper).

[General Device Description]: T The pediatric urine bag is composed of Bag.

[Intended use]: The pediatric urine bag used in storing the urine, For Clinical Use.

[Sterilization Method]: EO sterilization

[Validity]: 5 years.

[Use method]:

- 1. Open the single packaging**
- 2. Unroll the bag flat**
- 3. Remove the sticker and glue the bag on the baby's body (genital area)**
- 4. Start collecting urine drain into the bag**
- 5. When finished, gently remove the urine bag from baby's body**
- 6. Make sure not to spill any urine from the container**
- 7. Discard immediately after use**

[Cautions]:

- For single patient use only.
- Sterile if package is unopened or damaged.
- Store in a dry, cool and dark place.

[Storage and transportation]:

1. The device shall be stored at a relative humidity of less than 80%, the storage environment shall be kept non-corrosive gases, dry, avoiding sunlight, well-ventilated conditions, and clean.
2. The device shall be taken care during transportation and handling.

[Contact information]:

Changshu Kangbao Medical Appliance Factory

Furong Village, Baimao, Guli Town, 215500 Changshu City, Jiangsu Province,

Instruction for Use Pediatric urine bag			Prepared by	Li Xiuren
			Checked by	Zhou Ting
Doc. No.	KB/CE01-08		Approved by	Zhou Hongbao
Effective date	2022.03.01	Ver. B/0	Page No.	Page 2 of 2

PEOPLE'S REPUBLIC OF CHINA

TEL: +86-512-52539293 FAX: +86-512-52301322

Email: kb@kang-bao.cn Website: www.kang-bao.cn

The authorized representative of European Union:

MedPath GmbH

Mies-van-der-Rohe-Strasse 8,











80807 Munich, Germany

Phone: +49(0)89 189174474

Fax: +49(0)89 5485 8884

Email: info@medpath.pro

[Symbols]

	CE Mark		Lot number
	Single use		Date of manufacture
	Sterilized by EO		Manufacturer
	Notice		European union representative
	The product includes DEHP		Indicates the item is a medical device