
HU: Sebészeti arcmaszka
EN: Surgical Face Mask
DE: Chirurgische Gesichtsmaske
FR: Masque chirurgical
NL: Chirurgisch gezichtsmasker
IT: Maschera chirurgica per il viso
ES: Mascarilla quirúrgica

PT: Máscara Cirúrgica
SE: Kirurgisk ansiktsmask
DK: Kirurgisk ansigtsmaske
NO: Kirurgisk ansiktsmaske
FI: Kirurginen kasvonaamio
CZ: Chirurgická maska na obličej
PL: Chirurgiczna maska na twarz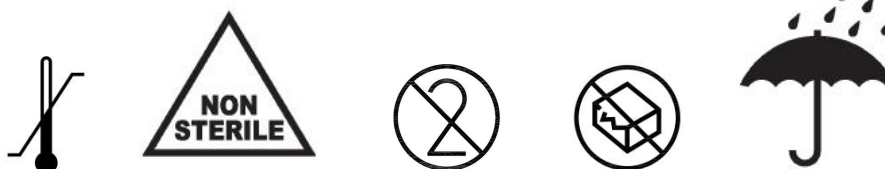

ENGLISH: INSTRUCTIONS FOR USE

APPLICATIONS: DIN EN 14683:2019/ISO 22609:2004 TYPE IIR SURGICAL MASK: Protection against non-toxic and low-to-average toxicity solid and liquid aerosols (e.g. oil-mists).

CHECK PRIOR TO USE: Inspect outer/inner layers and any component parts for holes, tears or damage prior to use. Should any damage be identified, DO NOT USE IT. Get a new one.

INSTRUCTIONS: 1. Hold the elastic loops of the mask. Make sure that the inner layer is facing you and the nose clip is on the upper side. 2. Stretch the ear loops to loop them around your ears. Adjust the mask until you feel a comfortable fit. 3. Cover your mouth and nose with the mask. Make sure to cover the bridge of the nose down to the jaw. 4. Lightly press and form the nose clip to create a good fit around the bridge of the nose.

WARNING: 1. Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of the product and result in sickness or death. 2. Do not use this product when oxygen concentration is less than 20%. 3. Leave work area and return to fresh air if breathing becomes difficult, or dizziness or other distress occurs. 4. Facial hairs or beards and certain facial characteristics may reduce comfort and breath ability. 5. No change or

modifications may be made to the mask. 6. Intended for single use only. No maintenance necessary. 7. Dispose of masks properly after use. 8. Store the masks in the packaging away from direct sunlight until use.

MEANING OF PICTOGRAMS/SYMBOLS: Storage (a) Temperature Range. (b) Non-Sterile Product. (c) Intended for Single use only. (d) Do not use when packaging is damaged. (e). Keep dry.

STORAGE & TRANSPORT: 1 Masks should be stored and transported in manufacturer's packaging, in the original plastic bags or cardboard boxes, in conditions with required humidity and temperatures. 2. Masks should be protected from direct sunlight, heat, chemical substances, moisture, dirt and mechanical damage.

Manufactured/Imported by: Refer to packaging for contact details.

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